

Prophylaxis Powder instructions

PRODUCT NAME: Prophylaxis Powder

PRODUCT MODEL: Classic

SPEC: 130g/bottle, 260g/bottle

INTENDED USE

Used in the oral cavity to grind and polish tooth tissue or prostheses to make their surfaces smooth and uniform, and remove plaque (tartar). This product is used with scaling and air polishing device, which is non-sterile and non-reusable.

PROPERTIES OF PRODUCT: White powder.

STRUCTURE AND COMPOSITION

It is composed of sodium bicarbonate, silicon dioxide, food flavoring essence (lemon flavor essence: citral, peach flavor essence: γ -Unsecalactone (peach aldehyde), fruit flavor essence: vanillin, ethyl vanillin, grape flavor essence: methyl anthranilate, cherry flavor essence: benzaldehyde, amyl butyrate, benzyl alcohol, anisaldehyde, strawberry flavor essence: davana oil, ethyl 3-methyl-3-phenylglycidate).

STORAGE, TRANSPORTATION CONDITIONS AND SHELF-LIFE

1. Storage conditions: temperature: - 10 °C to 40 °C, relative humidity: 10% to 95%, sealed storage.
2. Please transport this product under environmental conditions with a temperature of - 10 °C to 40 °C and a relative humidity of 10% to 95%.
3. Shelf-life: Under specified storage conditions, the shelf-life is 24 months.
4. Shelf-life after bottle opening: After bottle opening, the bottle cap needs to be tightly closed, and the expiration date after bottle opening under specified storage conditions is 3 months.

INSTRUCTION

Load the prophylaxis powder into the powder chamber of the scaling and air polishing device, and operate according to the operation instructions of the applicable device.

PRECAUTIONS

1. Prophylaxis powder shall not be stored near acidic solutions or heat sources, and shall not be directly exposed to the air. If caking, discoloration and other abnormal phenomena occur, it shall be stopped using.
2. Users and patients have to wear eye protection, and users have to wear gloves and mask.
3. Apply Vaseline® to the patient's lips before treatment.
4. In case the prophylaxis powder comes into contact with the eyes during treatment, it should be rinsed with a lot of water. Do not swallow it during the treatment.
5. Please use the high-speed evacuator of your dental unit.
6. Prophylaxis powder shall be disposed and destroyed according to relevant local regulations when it expired, and shall not be discarded in drainage ditches or rivers or mixed with household garbage.
7. In addition to people with contraindications, the product is suitable for adults and children who need oral

treatment.

8.This product is for oral cavity use only, and cannot be reused or used orally. Do not use after expiration date.

CONTRAINDICATIONS

1.Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

2.Predisposed persons may be sensitive to powder. If allergic reactions are observed, stop using the product and completely remove the powder from mouth.

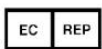
3.The treatment of deep periodontal pockets can cause bacteraemia. For the following risk patients (endocarditis, pregnant women, breast feeding, contact infectious diseases), immune deficiency (neutropenia, granulocytopenia, diabetes, hemophilia), and undergo treatment (radiotherapy, chemotherapy, antibiotic treatment), Please apply appropriate restrictions.

4.Do not direct the jet of powder toward fillings, crowns or bridgework as this could damage these restorations.

5.Patients with severe periodontal disease are advised to diminish inflammation before air polishing treatment.

APPLICABLE INSTRUMENT

Suitable for scaling and air polishing devices. It's recommended to use the devices produced by our company.

EXPLANATIONS FOR SYMBOLS									
	Manufacturer's logo		Manufacturer		Use-by date		Date of manufacture		Batch Code
	Shake well before use		Consult instructions for use		Temperature limit		Fragile, handle with care		CE marking
	Humidity limitation		Stacking limit by number		Keep dry		Keep away from sunlight		This way up
	Medical device		Unique device identifier			Authorised Representative in the European Community			

BASIC INFORMATION

Document No.: DQP050000055

Version: 07/2023-(n)

Date of compilation: 2023/07/03

 Guilin Veirun Medical Technology Co., Ltd.
Address: No. D-07 Information Industry District, High-Tech Zone,
541004 Guilin, Guangxi, PEOPLE'S REPUBLIC OF CHINA
Tel: +86(773)5880619 Fax: +86(773)2260519
Web: www.veirun.com E-mail: info@veirun.com
SRN: CN-MF-000035683

Shanghai International Holding Corp. GmbH (Europe)
Address: Eiffestrasse 80, 20537 Hamburg, Germany
Tel: +49-40-2513175/+49 163 6233205 E-mail: shholding@hotmail.com
SRN: DE-AR-000000001

